

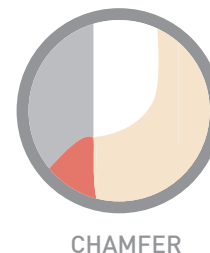
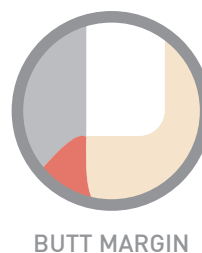
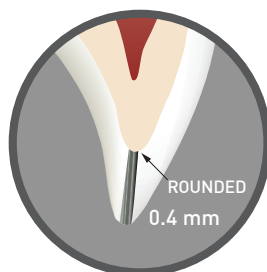
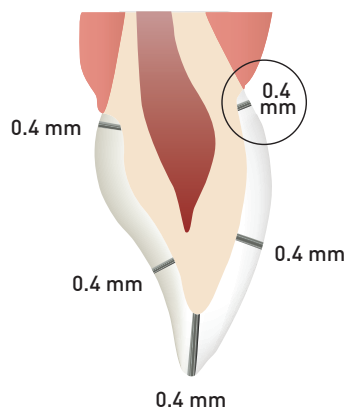
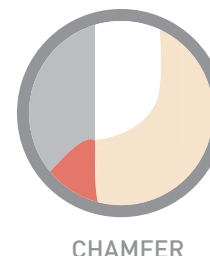



ANTERIOR CROWN

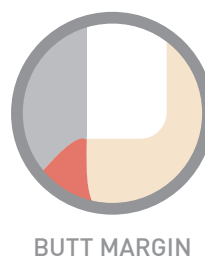


POSTERIOR CROWN

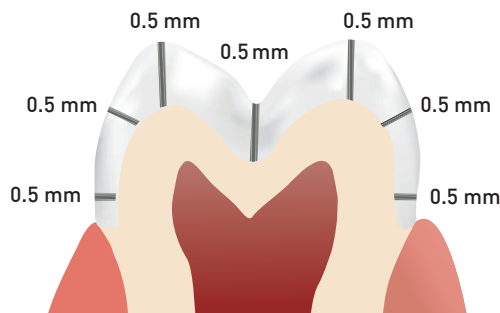


INLAY/ONLAYS

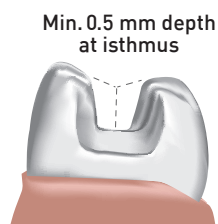

Min. 0.5 mm wide at isthmus

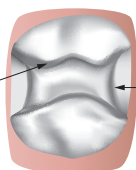

0.5 mm prep floor at gingival

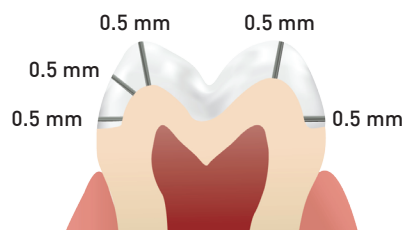

BUTT MARGIN



301 S 1st St, Champaign, IL 61820




1-800-742-3629



ragledental.com

ANTERIOR AND POSTERIOR BRIDGES

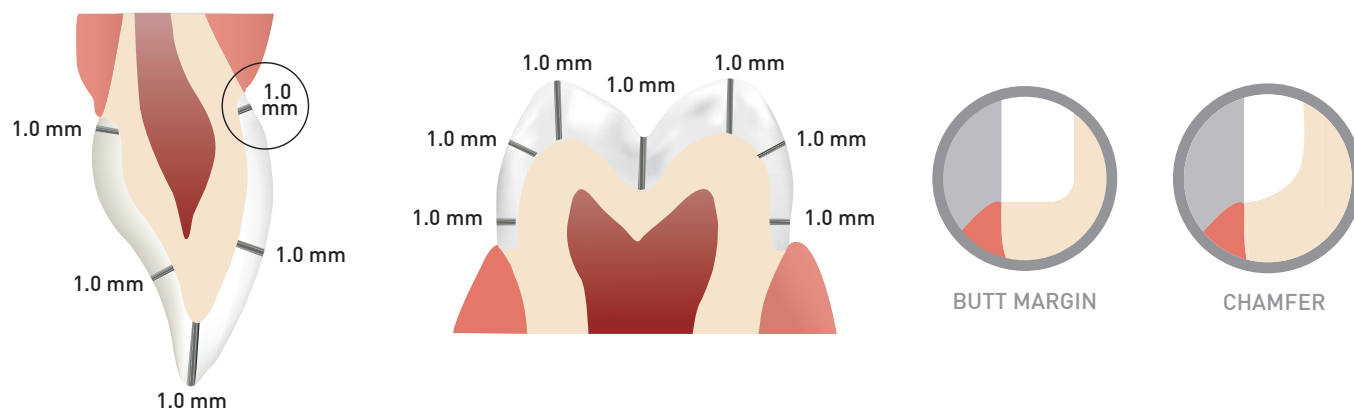

INDICATIONS

ANTERIOR	CERA-LEUX™ HSZ
Crown	✓
Bridge (up to full arch)	✓

POSTERIOR	CERA-LEUX™ HSZ
Crown	✓
Bridge (up to full arch)	✓
Inlay	✓
Onlay	✓

CEMENTATION

Conventionally cement following the manufacturer's instructions for use with zirconia restorations.

CHAIRSIDE ADJUSTMENTS

If chairside adjustments are needed:

- Fine diamond tools are recommended using a water-cooled high speed hand piece.
- Use light pressure to avoid overheating and micro-cracking of sintered body.
- Polish with diamond impregnated rubber polisher; maintain minimum wall thickness and connector cross sections.



301 S 1st St, Champaign, IL 61820



1-800-742-3629



ragledental.com